

Ocufresh Night

Lubricating Eye Ointment

5g

Preservative Free

What is Ocufresh Night?

Ocufresh Night is a preservative-free, smooth, off-white to pale yellow, free from foreign matters, sterile ointment for use in severe dry eye symptoms.

Composition:

Liquid paraffin 42.5% w/w, white soft paraffin 57.3% w/w and wool alcohol (or lanolin alcohol) 0.2% w/w.

What is Ocufresh Night used for?

Ocufresh Night is used for night-time relief of dry eye sensations (severe dry eyes).

The product provides strong, soothing, lubricating and moisture-retention qualities with long-lasting relief from the symptoms of severe dry eyes such as redness, discomfort, soreness, burning, and gritty sensation; by acting as a lubricant and as a barrier against moisture loss. Ideal for use at night, just before bedtime.

Target population group:

The product is targeted for use by individuals of all ages and gender (both male and female), suffering from severe dry eyes, after consultation with a doctor/physician or optician if use is required for more than 30 consecutive days. It may be also used during pregnancy and breast feeding.

How much to use:

Apply a thin line of ointment about half an inch (1 centimeter), placed into each affected eye(s) by gently squeezing the tube. Use, as required for relief of symptoms, but preferably before sleep.

Consult a doctor/physician or optician if use is required for more than 4 times daily or more than 30 days.

How to use:

Before opening a new tube of Ocufresh Night Ointment, please check that the safety strip on the cap of the tube is unbroken.

- Wash and dry your hands well before use.
- Tear of the safety strip to break the seal.
- Remove the outer cap.
- If you are a contact lens wearer, remove your lenses.
- Tilt your head back and look up.
- Gently pull down the lower lid of the eye to form a small pocket between your eyelid and your eye.

- Hold the tube between the thumb and middle finger of the other hand, turn the tube upside down near to the eye, and try not to touch the eye with the nozzle.
- Apply enough pressure to the tube to release a thin line of ointment about half an inch (1 centimetre), or as directed by your doctor, Pharmacist or optician.
- Blink your eyes a few times.
- If you missed the eye, then repeat the above steps until you have enough line of ointment on your eye.
- Carefully wipe away any excess ointment around the eyelid with a tissue.
- Repeat the process for the other eye, if required.
- Replace the outer cap on the container, do not touch the applicator tip with anything, including your eye or your fingers.

Precautions and warnings:

- Do not use in case of eye infection, injury or corneal abrasion.
- Do not use if the tube is damaged.
- Do not use the product if allergic to wool (lanolin) alcohol or any of the other ingredients.
- Do not use the product after the expiry date or 1 month after first opening.
- Do not use the product with contact lenses.
- Do not share this product with others; it should be used only by the concerned user.
- In case of any additional local ocular administration, there should be an application interval of at least 30 minutes between the use of the two products; the eye ointment should be the last product instilled.
- Do not touch the eye or any other surface with the tip of the tube in order to avoid scratching the eye or contaminating the product.
- Replace lid (cap) gently after using the tube; do not overtighten the cap.
- For use in your eyes only, do not swallow.

- Keep out of the reach and sight of children.
- If the problem does not improve within 2 days or worsens at any time, consult your doctor/physician or optician.
- Consult your doctor/physician or optician if you are pregnant or breast-feeding.
- If you feel abnormal eye sensation, stinging or pain in the eyes after using this product, talk to your doctor/physician or optician.

How to store Ocufresh Night?

- Store below 30° C.
- Keep the tube in the outer carton to protect from light and moisture.
- Dispose of your empty or unused tube according to the local waste management regulations.

Possible side effects:

This product may cause:

- Mild burning sensation in the eye.
- Mild discomfort in the eye.
- Temporary blurring of vision. Do not drive or operate machinery until normal vision is restored.
- Rarely, allergic reaction.

Symbol	Explanation
	Manufacturer
	Distributor
	Medical Device
	CE mark
	Sterilized using aseptic processing techniques
	Do not use if the outer packaging is damaged
	Lot number
	Expiry date [YYYY-MM]
	Read instructions for use
	Do not store above 30°
	Shelf life after opening
	Caution

www.ocufresh.co.uk

BLU[▲]ONT

 Blumont Healthcare Ltd.
23 Moortown Close, Grantham,
NG31 9GG, United Kingdom
www.blumonthhealthcare.com

EU **REP**

MedNet EC-REP IIB GmbH
Borkstrasse 10,
48163 Muenster,
Germany
GO/DRUGS/176

MD

STERILE A

CE 2803

Date of last review: 01.11.2025
IFU1617A.01