

IFU1715.00

ocufresh®

Everyday

Lubricating eye drops

Sodium Hyaluronate 0.2%

Preservative Free

10 ml

Preservative-free, sterile, lubricating, moisturizing and protective, isotonic, biocompatible, buffered, ophthalmic solution with Sodium hyaluronate 0.2%. Recommended for sensitive, dry and dehydrated eyes. Suitable also with contact lenses inserted, compatible with all contact lenses.

The absence of preservatives reduces the risk of adverse reactions and makes the product also suitable for therapies that require frequent and prolonged applications, such as for nocturnal corneal protection.

Indications

- Lubricates, moisturizes and protects sensitive, dry and dehydrated eyes;
- Provides relief in case of itching, burning, FBS (Foreign Body Sensation);
- Ideal in case of non-traumatic/non-pathological changes of the tear film;
- Ideal in case of environmental ocular stress (excessive exposure to sunlight, wind, dust, air conditioning, dry and polluted air, etc.);
- Ideal in case of ocular stress (extended computer use, unfavorable lighting conditions, night driving etc.);
- Indicated for prolonged use of contact lenses;
- Suitable for use whilst wearing contact lenses;
- Product compatible with homeopathic treatments.

Instructions for use

Instil one or two drops per eye, one or more times a day, as needed or as per your specialist advice. For contact lens users, instil 2 - 3 drops on the lens before or after use. Instil drops many times a day or as needed, even whilst wearing contact lenses.

Dispose according to the local laws.

Warnings

- In case of eye irritation, stop using the product and contact a specialist;
- Do not use in case of allergy or hypersensitivity to any of the product components;
- Keep away from heat sources, store in a dry place at temperatures between 5 and 35 °C;
- Keep out of sight and reach of children;
- Do not swallow;
- Do not use the product after the expiry date;
- Use within 180 days of first opening;
- Do not use the product if the bottle appears damaged;
- Do not let the tip of the bottle touch your eye or eyelid;
- Close the bottle firmly after each use;
- Administer the product at least 5 minutes before or after the administration of other topical ophthalmic products.

Composition

Sodium hyaluronate 0.2%, Disodium edetate, isotonic solution buffered to pH 7.2.

Does not contain

Preservatives, Benzalkonium chloride, Chlorhexidine, Parabens, Phenoxyethanol, Thimerosal, and components from animal origin.

Distributed by

BLUMONT

Blumont Healthcare Limited.
23 Moortown Close, Grantham,
NG31 9GG, United Kingdom.

www.ocufresh.co.uk

CE
0051

STERILE A

5°C 30°C