



Everyday

Eye Drops

0.5ml

**Singles - Sodium Hyaluronate 0.2%**

Ocufresh Everyday Eye Drops – Singles is a sterile, ophthalmic solution, with physiological pH and pharmaceutical grade Hyaluronic Acid (as sodium hyaluronate), a polymer normally present in different areas of the human body. Hyaluronic acid has viscoelastic, muco-mimetic and muco-adhesive properties that improve tear film stability and maintain ocular surface lubricated and hydrated.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

Ocufresh Everyday Eye Drops – Singles is compatible with soft and rigid contact lenses.

Indications

- Ocufresh Everyday Eye Drops - Singles moisturizes and lubricates eye relieving symptoms of irritation, dryness and foreign body sensation caused by:
 - Dry eye syndrome;
 - Mechanical factors (contact lenses use, after eye surgery);
 - Environmental factors (wind, smoke, pollen, dust, pollution);
 - Visual stress (excessive light, prolonged use of computers, PC and tablets).

Instructions for use

- Wash and dry your hands thoroughly before use.
- Open the protective aluminium pouch.
- Detach the single dose container from the strip and open the single-dose container by turning and then lifting off the closing cap.
- Tilt your head back and look up.
- Gently pull down the lower lid of the eye to form a small pocket between your eyelid and your eye.
- Hold the single-dose container into other hand, turn the container upside down and apply slight pressure to the single-dose container to release 1 drop of Ocufresh Everyday Eye Drops – Singles into the pocket.
- If you think that you have missed the eye, then insert another drop.
- Repeat the process for the other eye, if required.
- After use immediately close the single-dose container with the cap; the product can be used up to 24 hours after first opening. In order to close it the cap must first be turned over.
- The product can be used as required or according to the advice of your doctor.

Warnings and Precautions

- Product is for ophthalmic use only;
- Product is for single patient use;
- Do not use the product if the aluminum pouch is damaged or if the single dose is not perfectly closed;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch eye or any other surface with the container's dispensing edge;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Do not use the product after the expiry date indicated on the package;
- Do not use in case of known incompatibility towards any of the ingredients;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- Dispose of the product according to your local waste management regulations;

- Keep out of the reach and sight of children;
- The product is usually intended for continuous use for no more than 30 days; if symptoms persist consult your doctor;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- Ocufresh Everyday Eye Drops – Singles is not indicated for treatment of infection and other ocular diseases. These conditions require medical attention;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

Possible side effects

- Immediately after instillation may cause a slight temporary blurring of vision due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before any activity that requires clear vision.

Composition

- Sodium Hyaluronate 0,2%, Citric Acid Monohydrate, Trisodium Citrate Bihydrate, Sodium Chloride, Water For Injection.
- Does not contain preservatives.

Package

30 re-closable single dose of 0.5 ml.

Storage condition

- Store between 2°C and 30°C.
- The single dose content must be used within 24 hours after the first opening.
- Store unused single doses in the aluminum pouch until needed.

Validity

See the expiry date reported on the box and on the single-dose container.

www.ocufresh.co.uk


BLUMONT
Blumont Healthcare Limited.
23 Moortown Close, Grantham,
NG31 9GG, United Kingdom.



DT/01/DT/006