

ocufresh®

Lubricating eye drops

Hypromellose 0.3%

Ocufresh Lubricating eye drops preservative-free is a hypromellose-based, sterile, soothing and emollient ophthalmic solution, with a pH similar to natural tears.

Hypromellose (HPMC) is a viscous polymer with wetting and muco-mimetic properties which can stabilize the tear film on the surface of the eyes by creating a protective, transparent and viscoelastic shield. Hypromellose protects and lubricates the ocular surface reducing discomfort and providing soothing comfort to irritated eyes with long-lasting relief. It helps to restore the natural tear film for lasting relief.

Thanks to its specially developed preservative-free closure system, Ocufresh Lubricating eye drops does not need preservatives. The product is compatible with contact lenses.

Indications

- For relief from symptoms of dry eyes such as burning, itching and foreign body sensation.
- It also acts as a lubricant and moistener in dry eye conditions, including those associated with the use of PC, infrequent blinking, certain medical treatments, atmospheric pollution, dry atmospheres (e.g. air conditioning, wind and sun) and ophthalmic stress due to the use of contact lenses.
- Ocufresh Lubricating eye drops preservative-free can be used in case of moderate dry eye syndrome or dry eyes symptoms or more commonly, in case of discomfort sensation.

Instructions for use

- Tear off the tamper-evident seal before first administration.
- Remove the protective cap and open the bottle.
- Instill 1-2 drops in each eye slightly pressing the bottle.
- Shake the bottle to remove any residual drop before replacing the cap on the container, trying not to touch the applicator tip with anything, including your fingers.
- After use, close the bottle: the content should be re-used within 3 months from first opening.
- The product is suitable for frequent use. Unless your doctor has told you differently, you can use the product several times a day (3 or 4 times or more often if required) to provide sufficient symptoms relief.

- The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the special preservative-free closure system.

Warnings and Precautions

- This product is only for use in eyes.
- Remove contact lenses before using this product.
- The product is intended for use only by the concerned patient.
- Do not use the product if the container is damaged or not perfectly sealed when you open it for the first time.
- During the application of the eye drops, do not touch your eyes or any other surface with the dropper of the container.
- Do not use in case of known intolerance/allergy to any of the ingredients.
- The administration of other eye medicines at the same time as this product should be carried out under the supervision of your doctor who can evaluate possible interactions; there should be an interval of at least 15 minutes between the application of any other eye medicine and this product.
- In case of problems during treatment, stop using this product and consult a doctor.
- Check the expiry date on the box before use; do not use the product after the expiry date indicated on the package. After use, dispose of the product according to local regulations.
- Keep out of sight and reach of children.
- Do not swallow.
- The product is usually intended for continuous use, no more than 30 days.
- Consult your doctor if symptoms persist.
- Pregnant or breast-feeding women and those who have recently undergone eye surgery must consult their doctor before use.

Ocufresh Lubricating eye drops preservative-free is not indicated for treatment of infection and other ocular diseases. These conditions require medical attention.

Possible side effects

- Vision may be temporarily blurred after use of the product.
- Also, minor burning/stinging/irritation may temporarily occur as a consequence of the ocular irritation.
- If any of these effects persist or worsen, tell your doctor promptly.

Composition

- Hydroxypropylmethylcellulose (HPMC), sodium phosphate monobasic monohydrate, sodium phosphate dibasic dodecahydrate, sodium chloride, Water for Injections.
- Does not contain preservatives.

Package

- 10 ml multi-dose bottle, preservative free.

Storage conditions

- Do not store above 30°C.
- After the first opening, the contents of the bottle must be used within 3 months.

Rev. 05.01 of 01/2021

www.ocufresh.co.uk

Distributed by

BLUMONT

Blumont Healthcare Limited.

23 Moortown Close, Grantham,
NG31 9GG, United Kingdom.

CE

0546
MEDICAL DEVICE

STERILE A