

IFU1716.00



Intense Relief

Advanced Dry Eyes eye drops

Sodium Hyaluronate 0.4%

Preservative Free

10 ml

Preservative-free, sterile, lubricating, moisturizing, protective and antioxidant, isotonic, biocompatible, buffered, ophthalmic solution with Sodium hyaluronate 0.4%, Alpha lipoic acid and Vitamin E-TPGS. Recommended for dry, dehydrated and tired eyes.

The absence of preservatives reduces the risk of adverse reactions and makes the product also suitable for therapies that require frequent and prolonged applications, such as for nocturnal corneal protection.

Indications

- Lubricates, moisturizes and protects dry, dehydrated and tired eyes;
- Provides relief in case of itching, burning, FBS (Foreign Body Sensation);
- Stabilizes and reintegrates the tear film in cases of corneal-conjunctival histological alterations caused by oxidative stress and accumulation of free radicals on the ocular surface;
- Contributes to the protection of the cornea and conjunctiva by reducing the risk of inflammation and assisting the reparative processes;
- Ideal in case of environmental ocular stress (wind, dust, air conditioning, dry and polluted air, etc.);
- Ideal in case of ocular stress (extended computer use, unfavourable lighting conditions, night driving etc.);
- Suitable also for use while wearing contact lenses;
- Compatible with homeopathic treatments.

Instructions for use

Instil one or two drops per eye, one or more times a day, as needed or as per your specialist advice. For contact lens users, instil 2 - 3 drops on the lens before or after use. Instil drops many times a day or as needed, even whilst wearing contact lenses.

Dispose according to the local laws.

Warnings

- In case of eye irritation, stop using the product and contact a specialist;
- Do not use in case of allergy or hypersensitivity to any of the product components;
- Keep away from heat sources, store in a dry place at temperatures between 5 and 35 °C;
- Keep out of sight and reach of children;
- Do not swallow;
- Do not use the product after the expiry date;
- Use within 180 days of first opening;
- Do not use the product if the bottle appears damaged;
- Do not let the tip of the bottle touch your eye or eyelid;
- Close the bottle firmly after each use;
- Administer the product at least 5 minutes before or after the administration of other topical ophthalmic products.

Composition

Sodium hyaluronate 0.4%, Alpha lipoic acid, vitamin E-TPGS, isotonic solution buffered to pH 7.2.

Does not contain

Preservatives, Benzalkonium chloride, Chlorhexidine, Parabens, Phenoxyethanol, Thimerosal, and components from animal origin.

Distributed by



Blumont Healthcare Limited.
23 Moortown Close, Grantham,
NG31 9GG, United Kingdom.

www.ocufresh.co.uk



0051

