



Intense Relief

Eye Drops

0.5ml



Singles - Sodium Hyaluronate 0.4%

Ocufresh Intense Relief Eye Drops - Singles is a sterile ophthalmic solution with a physiological pH, based upon a patented technology of cross-linking Hyaluronic Acid (HA) with Urea, therefore enhancing and improving the specific mechanical properties of native HA, resisting to hyaluronidase degradation much longer than native HA, thus with higher and prolonged lubricant and hydrating action than native HA.

The hydrating web structure of cross-linked HA is capable to deliver water for a prolonged period improving the lubricating properties of native HA while boosting its moisturizing effects at the same time, it is therefore indicated as an adjuvant in dry-eye syndrome.

The cross-linked HA provides to the product lubricating, moisturizing and repairing properties, capable of giving relief to ocular dryness due to dry eye syndrome, irritated or tired eye in case of inflammatory conditions or environmental factors.

The formula was developed to ensure maximum comfort already in the first moments after the application. It is suitable for everyday use, daily and night-time.

It does not contain preservatives and/or molecules with anti-inflammatory and vaso-constrictive actions.

Ocufresh Intense Relief Eye Drops - Singles is compatible with soft and rigid contact lenses.

Indications

- For reinstatement and improvement of tear film stability in dry eye syndrome;
- After refractive surgery;
- Relieves redness, fatigue and discomfort of the eye or the ocular mucosa due to:
 - Environmental factors (wind, smoke, pollution, air-conditioned, dried salt, sea water, pool water and solar radiation);
 - Eye dryness due to excessive fatigue caused reading, studying and working on the computer for lengthy periods;
 - Lacrimal abnormalities secondary to inflammations, chronic blepharitis;
 - Lacrimal abnormalities secondary to eye surgery of various nature or eyelid aesthetic surgery;
 - Wearing soft or semirigid contact lenses.

Instructions for use

- Wash and dry your hands thoroughly before use;
- Detach the single dose container from the strip;
- Twist and remove the cap to break the seal;
- Place the cap taking care to put it on a clean surface;
- Instill 2 drops in each eye by gently squeezing the single dose, unless otherwise prescribed;
- After use immediately close the single-dose container with the cap: the product can be used up to 12 hours after first opening.

Warnings and Precautions

- Product is for ophthalmic use only;
- Product is for single patient use;
- Do not use the product if the single-dose container is damaged or unsealed when first purchased;
- Do not use if are visible particles and/or changes in the color of the solution;
- During application, do not touch eye or any other surface with the container's dispensing edge;
- In case of side effects during use, interrupt the treatment and consult a doctor;
- Do not use the product after the expiry date indicated on the package;
- Do not use in case of known incompatibility towards any of the ingredients;
- No interactions with drugs or other eye drops are reported; however, leave at least 15 minutes between administrations of any other eye drops;
- Dispose of the product according to your local waste management regulations;

- Keep out of the reach and sight of children;
- Consult your doctor before you use this product if you are pregnant or breast-feeding;
- For external use, do not swallow;
- It is necessary to report any serious incident occurred in relation to the device to the manufacturer and the Competent authority of the Member state in which you are established.

Possible side effects

Immediately after instillation may cause a slight temporary blurring of vision due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before any activity that requires clear vision.

Composition

- Sodium Hyaluronate Crosspolymer Aqueous Solution (of which Sodium Hyaluronate Crosspolymer 0,4%), Trometamol, Hydrochloridric acid 2N, Sodium chloride, Water for Injections.
- Does not contain preservatives.

Packaging

30 re-closable single dose of 0.5 ml.

Storage condition

- Store between 2°C and 30°C in a dry place away from heat and direct exposure to sunlight.
- The single dose content must be used within 12 hours after the first opening.

Validity

See the expiry date reported on the box and on the single-dose container.

Rev. 02.02 of 10/2023

www.ocufresh.co.uk



BLUMONT

Blumont Healthcare Limited.

23 Moortown Close, Grantham,
NG31 9GG, United Kingdom.

CE 0051

MD

STERILE A

#

DT/01/DT/010