

Natklor®

Sodium Chloride 5% Eye drops

For the temporary relief of corneal oedema

- Preservative free, sterile, hypertonic, biocompatible, buffered, ophthalmic solution with Sodium chloride 5% and Sodium hyaluronate.
- The absence of preservatives reduces the risk of adverse reactions and makes the product also suitable for therapies that require frequent and prolonged applications, such as for nocturnal corneal protection.
- Product compatible with homeopathic treatments.

Indications

For temporary relief from the signs and symptoms of corneal oedema.

Instructions for use

- Wash and dry your hands carefully;
- Instil 1-2 drops per eye one or more times a day, as needed or as per your specialist advice;
- In case of use on children the product must be administered by an adult.

Warnings

- The use of the product may induce burning and temporary irritation. In case the problem persists or in case of ocular irritation, please consult your specialist;
- Do not use if you have an allergy or hypersensitivity to any of the product components;
- Consult your doctor for use in children below 3 years old, during pregnancy, breastfeeding and in presence of other pathologies;

- Do not use the product in case of ocular injuries;
- Keep away from heat sources, store in a dry place at temperatures between 5 and 35°C;
- Do not freeze;
- Keep out of the reach of children;
- Do not swallow;
- Do not use the product after the expiry date;
- Use within 180 days of first opening;
- Close the bottle firmly after use;
- Do not use the product if the bottle appears to be damaged;
- Do not let the tip of the bottle touch your eye or eyelid;
- The product is not compatible with contact lenses;
- Administer the product at least 5 minutes before or after the administration of other topical ophthalmic products;
- Dispose according to the local laws.

Composition

Sodium chloride 5%, Sodium hyaluronate, hypertonic solution buffered to pH 7.2.

Does not contain

Preservatives, Benzalkonium chloride, Chlorhexidine, Parabens, Phenoxyethanol, Thimerosal, and components of animal origin.



Distributed By

BLUMONT

Blumont Healthcare Limited.

23 Moortown Close, Grantham,
NG31 9GG, United Kingdom.

www.blumonthealthcare.com

STERILE A



Carton and Instructions leaflet are recyclable